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☞ Sterile Adhesive Dressing Aqua Protect
 For covering surface skin wounds. For absorbing low to moderate amounts of wound fluid. For protecting minor post-operative wounds.

For non-professional use
 For external use. The sterile adhesive dressing covers the wound and absorbs wound fluid.

Instructions for use

1. Clean and dry the wound and the surrounding area.
2. Remove the sterile pouch and release liner 1 and 2. Apply the dressing to the wound, creating a barrier on the surrounding skin. Do not touch wound pad or adhesive. Remove release liner 3. Press down the edges of the dressing.
3. Check the wound regularly and replace the dressing at least daily. Remove the dressing by slowly pulling it away from the skin lengthwise. An old dressing can be replaced with a new one until the healing process is completed.

⚠ Do not use the dressing if the sterile packaging has

been damaged or previously opened. Dressings should not be used when allergic to any of its components. Stop using in the event of skin irritation or infection. Consult a doctor if it persists for adequate infection treatment. Dressings should not be reused as this can cause infection. Dressing should not be used on wounds with much wound fluid as this can cause softening and breaking of wound tissue. Do not use if the damage of the wound extends below all layers of the skin. Do not use on third degree burns. Dispose of the bandage according the regulations and do not touch the absorbent wound pad.

Report serious incidents to the manufacturer and the competent authority.

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