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EN: Sterile non-woven compress Use the sterile non-woven compress for cleaning and covering surface wounds. The compress is absorbent, breathable and individually supplied in sterile packaging. For non-professional use. Sterile compress for external use. Mode of action: The compress covers the wound and absorbs wound fluid.

⚠️ Do not use the compress if it is dirty, or if the sterile packaging has been damaged or previously opened in error. Stop using in the event of skin irritation or infection and consult a doctor if it persists. Do not reuse the compress as this can cause infection.

Store in the original packaging as it contains important information. Report serious incidents to the manufacturer and the competent authority.

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EN: Instructions for use:

1. Carefully clean the wound and surrounding area.
2. Remove the sterile packaging and Place the compress over the wound, double-folded if required. Do not touch the side that comes into contact with the wound.
3. Attach the compress using a bandage or adhesive plaster.
4. Check the wound regularly and replace the compress at least daily.

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